

# GLOBEX

## EVACUATION SOLUTIONS

ESTABLISHED 2010

RAISING AWARENESS - IMPROVING STANDARDS

# GLOBEX STANDARD EVACUATION CHAIR



DESIGNED AND  
MANUFACTURED IN THE UK

# THANK YOU FOR CHOOSING GLOBEX

## INTRODUCTION

The GLOBEX Standard Evacuation Chair is a Class 1 Medical Device designed to transport non ambulant persons downstairs during an emergency or when lifts can not be used. It is intended for use by trained competent operators only.

The responsible person must ensure that suitable and sufficient emergency procedures, equipment, training and maintenance is in place to ensure the safe evacuation of all users of the premises whenever it is occupied without depending on the emergency services.

This manual should be retained for reference and kept in the rear of the headrest. The GLOBEX Standard Evacuation Chair comes complete with:

- 1 x Dust Cover
- 2 x Wall mounting brackets (attached to the chair)

Upon receipt of your GLOBEX Standard Evacuation Chair all packaging and the wall mounting brackets attached to the chair should be removed. If not commissioned on site by a GLOBEX approved technician an inspection should be carried out to ensure:

- The chair has not been damaged in transit
- All components are complete and in good condition
- All nuts and bolts are secure
- Ski mechanisms open
- All wheels operate correctly
- Head and waist belts operate correctly
- Mechanisms easily open, close and engage as required
- Ski belts rotate freely around the guides

## WALL MOUNTING

The chair should be wall mounted in a suitable location as specified by GLOBEX personnel or in accordance with the building fire risk assessment and emergency procedures. The brackets provided should be secured with suitable fittings, level and around 225mm apart. The crossbar of the ski tracks hooks onto the brackets when the chair is in the closed position. The dust cover provided should be fitted when the chair is not in use.

## GENERAL INFORMATION





1. Your GLOBEX Standard Evacuation Chair must only be used by trained operators
2. Training should be conducted by authorised GLOBEX personnel only
3. Refresher training should be undertaken at least annually
4. Two person operation is recommended
5. The operator must never leave a user in the chair unattended
6. The operator must never let go of the chair when travelling down the stairs
7. Maintenance should be conducted by authorised GLOBEX personnel only at least annually
8. To ensure the optimal performance, the ski belts should be rotated every 3 months

## GUARANTEE

GLOBEX Evacuation Chairs are guaranteed for five years against defective materials and workmanship. This may be extended with a GLOBEX Compliance Package. Parts subject to wear and tear or damage caused by vandalism or misuse are excluded.

Maintenance should only be carried out by GLOBEX approved personnel at least annually to ensure compliance with GLOBEX requirements and in the UK, the Provision and Use of Work Equipment Regulations 1998 (PUWER)

<b>SAFE WORKING LOAD:</b>	182Kg / 28.6st / 400lbs
<b>STAIR TYPE / ANGLE:</b>	Straight /28 40 Degrees
<b>DIMENSIONS STORED:</b>	Height 1050mm Width 520mm Depth 210mm
<b>DIMENSIONS DEPLOYED:</b>	Height 1380mm Width 520mm Depth 810mm
<b>WEIGHT:</b>	9.5Kg

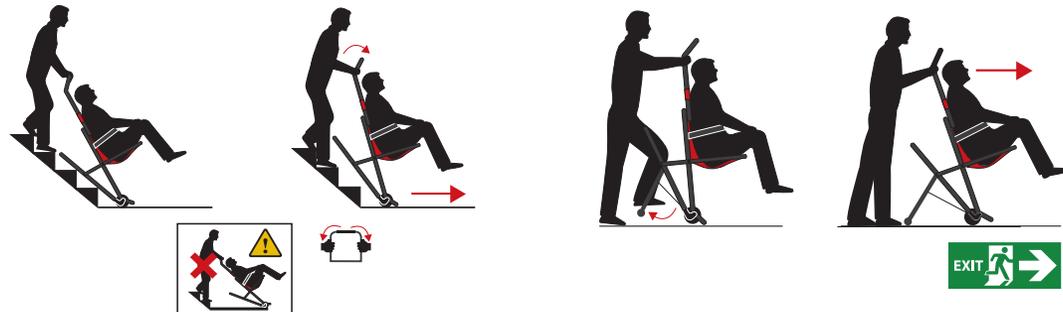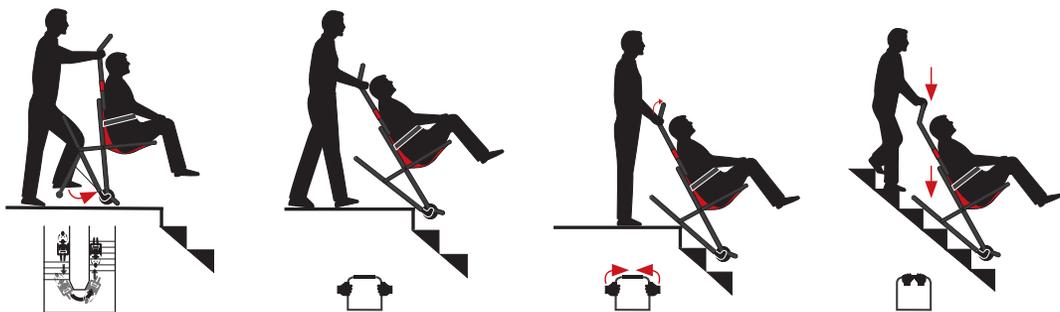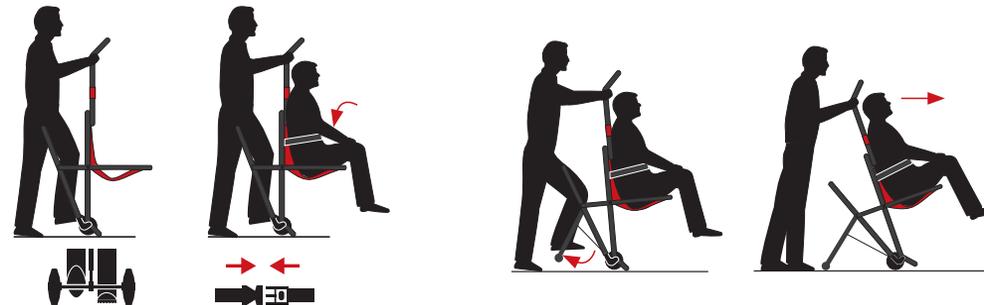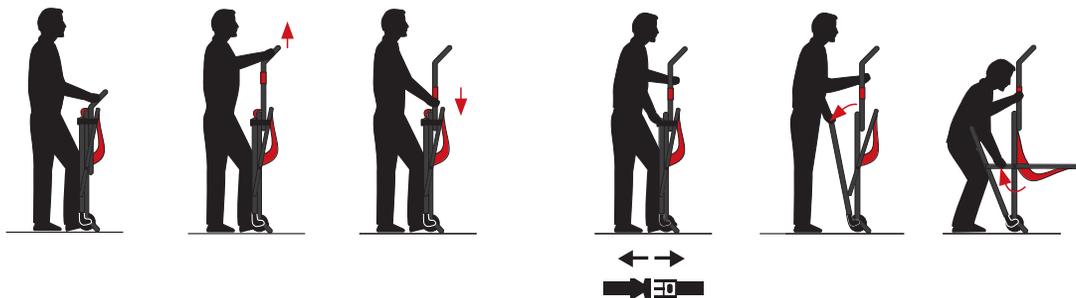

MUST ONLY BE USED BY TRAINED PERSONNEL ONLY



# GLOBEX

## EVACUATION SOLUTIONS

ESTABLISHED 2010

RAISING AWARENESS - IMPROVING STANDARDS

SURVEYS • DEMONSTRATIONS • SUPPLY • INSTALLATION • TRAINING • MAINTENANCE • CONSULTANCY • COMPLIANCE PACKAGES

For all UK requirements 0800 980 3473

NATIONAL & INTERNATIONAL SERVICE

Globex Evacuation Limited - Unit 1 Hitch's Yard, Church Street, Ware, Hertfordshire, SG12 9ES, UK  
Sales: 01920 449123 | Service: 01920 449020 | Accounts: 01920 449023 | E: [info@globexevacuation.com](mailto:info@globexevacuation.com)

[www.globexevacuation.com](http://www.globexevacuation.com)